

E-Z Freezin[®] Packet Stand



USER GUIDE



ANIMAL REPRODUCTION SYSTEMS



14395 Ramona Ave. Chino, CA 91710

www.arssales.com ♦ sales@arssales.com ♦ TEL: (800) 300-5143 ♦ FAX: (909)597-3043

©2006 DUPREE, INC

EZF-PS-UG A



Part No. EZF-PS

Introduction

The E-Z Freezin[®] Packet Stand is designed to provide a convenient and stable support for up to three E-Z Freezin[®] packets and two centrifuge tubes during the freezing process. Detailed instructions for the preparation and freezing of stallion semen can be found starting on page 99 of the book *Techniques for Collection, Evaluation and Insemination of Stallion Semen, 3rd Edition*, which is available from ARS (P/N 537-342C).

Instructions:

1. Place estimated number of thawed packets in the E-Z Freezin[®] Packet Stand.



2. Place up to two centrifuge tubes (P/N 537-742/25) in rack.



3. Pipette desired amount of E-Z Freezin[®] into centrifuge tube containing the pellet.



Cleaning:

The E-Z Freezin[®] Packet Stand may be cleaned using warm soapy water. Rinse thoroughly and allow to dry. This item is not dishwasher safe.

WARNING: Do Not Autoclave!

LIMITED LIABILITY STATEMENT

DUPREE, INC., a corporation ("SELLER" or "DUPREE"), warrants to the original purchaser only that the products sold by its Animal Reproduction Systems Division shall be free from defects in material and/or workmanship WHEN DELIVERED BUT NOT THEREAFTER. The purchaser must examine such products immediately upon receipt thereof and determine whether the same conform to the description thereof as set forth in the product data sheet accompanying this Limited Warranty Statement. If any such product does not conform to any of the data sheet descriptive criteria provided by SELLER with this statement, or if any such product appears to be defective in any respect, such product shall be immediately returned to SELLER for replacement and/or repairs as appropriate to the product. Such inspection and return of any allegedly defective product shall be made within five (5) days from the date of delivery thereof to the original purchaser and, if not so returned, this WARRANTY shall be void and of no effect whatsoever. Any allegedly defective product returned for repair or replacement within said five day period will be repaired or replaced and reshipped to the original purchaser within ten (10) days after return of the defective product to SELLER.

The restrictive and limited nature of SELLER'S warranty obligations as herein set forth is required by reason of the character of the products and the circumstances and conditions under which the same are likely to be used. The warranty obligations of SELLER shall be limited to repair or replacement of

any allegedly defective product as set forth above and DUPREE shall not be liable or responsible under any circumstances or in any amount for incidental or consequential damages or for injury or damage to persons or property using or used in connection with any of such products, whether or not defective, or for loss of profits or other costs, expenses or charges of any kind whatsoever. SELLER'S warranty obligations as set forth above shall be absolutely void if the allegedly defective product is not returned to seller within the five day inspection period set forth above, or if the product is misused, neglected or damaged, or if the instructions for its use as set forth in the accompanying data sheet have not been strictly followed, or if anyone other than SELLER has made any attempt whatsoever to repair or to alter the product in any respect.

There are no warranties or representations by SELLER other than as set forth herein, either express or implied, as to any of the products or the origin thereof or any other matter relating thereto, and no person, firm or corporation is authorized to make any representation or to incur any obligation in the name or on the behalf of DUPREE, contrary to any of the provisions or conditions of this statement.

EXCEPT TO THE EXTENT EXPRESSLY PROHIBITED BY APPLICABLE LAW, THE FOREGOING LIMITED WARRANTY IS IN LIEU OF ANY AND ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO ANY WARRANTIES OF FITNESS OR MERCHANTABILITY.