

Collection Bottle Cover Warmer User Guide



The Collection Bottle Cover Warmer (P/N 562A-101) is designed to keep the Collection Bottle Cover (P/N CBC-101) at the appropriate temperature for collection of stallion semen. Properly warmed the Collection Bottle Cover will protect the spermatozoa from sunlight and climate.

Construction:

The Collection Bottle Cover Warmer is designed for both service and durability. The base of the Collection Bottle Cover Warmer is made from durable, molded plastic, with rubber feet to prevent the unit from slipping. A lighted switch, located on the base, indicates that the Collection Bottle Cover Warmer is on and operating. The aluminum cone is anodized and epoxy coated to resist corrosion. Inside, a porcelain socket holds a 40-watt light bulb which heats the unit. A thermostat regulates the temperature of the Collection Bottle Cover Warmer, insuring that it does not exceed 41°C (~106°F).

Directions for Use:

 Turn on the Collection Bottle Cover Warmer, and all other equipment, before assembling the Artificial Vagina.

- Place the Collection Bottle Cover over the cone of the Collection Bottle Cover Warmer, then prepare the A.V. for collection of semen.
- After the A.V. is properly assembled and filled with water, place the warmed Collection Bottle Cover over the Collection Bottle and attach it to the end of the A.V. case.

Alternate Use:

During normal use the Collection Bottle Cover may become wet or damp. If the Collection Bottle Cover is wet, it should be wrung out and allowed to air dry until damp. If the Collection Bottle Cover is damp, it may be placed on the Collection Bottle Cover Warmer for safe, fast drying.

CAUTION: Never place a wet Collection Bottle Cover on the Collection Bottle Cover Warmer, as this may result in water dripping through the vent hole at the top of the cone and onto the lamp. Water dripping onto the lamp or other internal components may cause permanent damage to the unit.



ANIMAL REPRODUCTION SYSTEMS

14395 Ramona Ave. Chino, CA 91710

www.arssales.com ❖ arssales@dupreeinc.com ❖ (800) 300-5143 ❖ FAX: (909) 597-3043

© 2002 DUPREE, INC. 562A-0086A

Maintenance:

The Collection Bottle Cover Warmer is supplied with a 40-watt light bulb. To replace the bulb, remove the four screws holding the flange of the aluminum cone to the molded plastic base. Carefully lift the cone to expose the bulb, remove the defective bulb and replace with a new 40-watt bulb. When changing the bulb, care should be taken not to break, accidently disconnect, or damage the wiring. After replacing the bulb, the cone should be rotated so that the surplus wire forms a spiral between the cone and the bulb. The wiring should never come in contact with the bulb.

<u>CAUTION:</u> Use only the recommended 40-watt bulb. Replacing the bulb with one of a higher power could damage the Collection Bottle Cover Warmer.

Cleaning:

The Collection Bottle Cover Warmer should be spot cleaned with warm soapy water. Never use a caustic cleaning product, as the plastic may be damaged.

WARNING: Never submerge this product in water, especially while it is plugged in, doing so could result in electrocution.







LIMITED LIABILITY STATEMENT

DUPREE, INC., a corporation ("SELLER" or "DUPREE"), warrants to the original purchaser only that the products sold by its Animal Reproduction Systems Division shall be free from defects in material and/or workmanship WHEN DELIVERED BUT NOT THEREAFTER. The purchaser must examine such products immediately upon receipt thereof and determine whether the same conform to the description thereof as set forth in the product data sheet accompanying this Limited Warranty Statement. If any such product does not conform to any of the data sheet descriptive criteria provided by SELLER with this statement, or if any such product appears to be defective in any respect, such product shall be immediately returned to SELLER for replacement and/or repairs as appropriate to the product. Such inspection and return of any allegedly defective product shall be made within five (5) days from the date of delivery thereof to the original purchaser and, if not so returned, this WARRANTY shall be void and of no effect whatsoever. Any allegedly defective product returned for repair or replacement within said five day period will be repaired or replaced and reshipped to the original purchaser within ten (10) days after return of the defective product to SELLER.

The restrictive and limited nature of SELLER'S warranty obligations as herein set forth is required by reason of the character of the products and the circumstances and conditions under which the same are likely to be used. The warranty obligations of SELLER shall be limited to repair or

replacement of any allegedly defective product as set forth above and DUPREE shall not be liable or responsible under any circumstances or in any amount for incidental or consequential damages or for injury or damaged to persons or property using or used in connection with any of such products, whether or not defective, or for loss of profits or other costs, expenses or charges of any kind whatsoever. SELLER'S warranty obligations as set forth above shall be absolutely void if the allegedly defective product is not returned to seller within the five day inspection period set forth above, or if the product is misused, neglected or damaged, or if the instructions for its use as set forth in the accompanying data sheet have not been strictly followed, or if anyone other than SELLER has made any attempt whatsoever to repair or to alter the product in any respect.

There are no warranties or representations by SELLER other than as set forth herein, either express or implied, as to any of the products or the origin thereof or any other matter relating thereto, and no person, firm or corporation is authorized to make any representation or to incur any obligation in the name or on the behalf of DUPREE, contrary to any of the provisions or conditions of this statement.

EXCEPT TO THE EXTENT EXPRESSLY PROHIBITED BY APPLICABLE LAW, THE FOREGOING LIMITED WARRANTY IS IN LIEU OF ANY AND ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO ANY WARRANTIES OF FITNESS OR MERCHANTABILITY.